REMOTE  
VIBRATION K-BALLS


[www.mlcreationco.com](http://www.mlcreationco.com)

Registered Design

Ball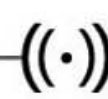

8 vibration modes  
1 vibration intensity

Button

Remote

Button

Made in China

RoHS

### **Button functions:**

- A. Press the button to turn on 'Vibration K-Ball' with the 1<sup>st</sup> vibration mode.
- B. After turning on, each press will change a vibration mode, after the 8<sup>th</sup> mode it will go back to the 1<sup>st</sup> mode.
- C. After turning on, anytime quick press twice will get into the auto cycle of 8 vibration modes, if want to get out the auto cycle of 8 vibration modes, just press once.
- D. Press the button for 1.2 seconds to turn off 'Vibration K-Ball'.
- E. The remote operation is same as the described above.

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## User manual

### Product features:

'Vibration K-Ball' is composed by 100% silicone and one environment-friendly metal ball, one environment-friendly plastic bullets. Daytime built into the vagina, not only strong and tightening abdominal muscles and genitals, you can enjoy the pleasure brought by remote vibration function anytime. It adopts USB charging. 1.5 hours of charging = 1 hour of use. It should be charged before the initial use, no function is available during the charging.

### Use:

Never use it at the swell or wound. Please stop the use in case of pain or discomfort. This product is used for adults only. Please place it out of the reach of children.

### Cleaning:

'Vibration K-Ball' is 100% waterproof.  
\* Clean it with antibacterial soap after each use.  
\* Do not use alcohol, acetone-based or gasoline-based cleaning products.

### Important tips:

It is only applicable to the adults aged 18 and above. Pregnant women or the patients with cardiac pacemaker, diabetes, phlebitis or thrombosis should consult a professional physician before using this product.

### Disclaimer:

Use this product at your own risk. Neither ML Creation nor its retailers will bear any legal liability arising out of the use.

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### Warning:

Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

### NOTE:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This device complies with Part 15 of the FCC Rules.

Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received, including interference that may cause undesired operation.